

# ColActive<sup>®</sup> PLUS Ag

## Collagen Matrix Dressing with Silver



### HOW DOES IT WORK?

ColActive<sup>®</sup> Plus Ag Collagen Matrix Dressing with Silver is an advanced wound care dressing made of collagen, sodium alginate, carboxymethylcellulose (CMC), ethylenediaminetetraacetic acid (EDTA) and silver chloride. The topically applied wound dressing will transform into a soft gel sheet when in contact with wound exudates. The dressing maintains a moist wound environment at the wound surface that aids in the formation of granulation tissue and epithelialization. The EDTA in the dressing removes zinc to inhibit the activity of matrix metalloproteinases (MMP), thus creating a suitable environment for wound healing. The silver chloride in the dressing provides effective broad spectrum antimicrobial activity. The dressings may be trimmed and layered for the management of deep wounds.

\*Covalon Technologies Ltd., Data on file.

Manufactured by:

**covalon**  
TECHNOLOGIES LTD.

[www.covalon.com](http://www.covalon.com)

### ✓ FACILITATES NATURAL WOUND HEALING

EDTA and collagen in ColActive<sup>®</sup> Plus Ag target and deactivate elevated MMP Activity\*.

### ✓ MOIST WOUND ENVIRONMENT

Alginate and CMC in ColActive<sup>®</sup> Plus Ag allow the dressing to absorb up to 20 times its weight in wound drainage, while still maintaining an optimal moisture balance for wound closure\*.

### ✓ ANTIMICROBIAL SILVER

ColActive<sup>®</sup> Plus Ag contains ionic silver for broad spectrum antimicrobial activity.

### ✓ CLINICAL TRACK RECORD

ColActive<sup>®</sup> Plus Ag has over ten years of clinical success in treating chronic stalled wounds.

### WHEN TO USE:

ColActive<sup>®</sup> Plus Ag is indicated for management of full and partial thickness wounds including: pressure ulcers, diabetic ulcers, ulcers caused by mixed vascular aetiologies, venous ulcers, donor and graft sites, abrasions, traumatic wounds healing by secondary intention, dehisced surgical wounds, first and secondary degree burns.

### WHEN NOT TO USE:

ColActive<sup>®</sup> Plus Ag dressings should not be used on third degree burns or on patients with a known allergy or sensitivity to animal derived collagen, silver, or any other ingredient.


Distributed by:




For ordering information call 800-243-2294  
or visit [www.hartmannusa.com](http://www.hartmannusa.com)

**HOW TO APPLY:**


1. Debride and irrigate the wound bed with an appropriate wound cleanser or distilled water before application.
2. Cut the dressing to fit the exact wound size. For heavily exudating wounds, apply the dressing directly to the wound bed. For dry wounds with very little exudate, moisten the wound bed with distilled water to begin the gelling process.
3. Cover the ColActive® Plus Ag with an appropriate secondary dressing.
4. Change dressings daily or as good nursing practice dictates.

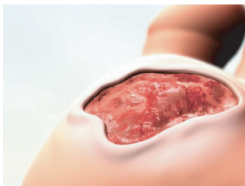

1.

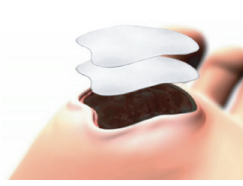

2.

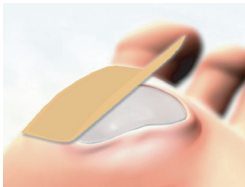

3.



4.

**CARE AND REMOVAL:**

1. Remove the secondary dressing with care. Gently remove ColActive® Plus Ag ensuring that the dressing removal does not damage any newly formed tissues.
2. Use an appropriate wound cleanser or distilled water to cleanse the wound site prior to applying a new ColActive® Plus Ag dressing, as per application instructions.

**AVAILABILITY:**

COLACTIVE® PLUS AG			
Collagen Matrix Dressing with Silver			
Reference Number	HCPCS	Size (in)	Dressings per Carton / Cartons per Case
1033 0000	A6021	2in x 2in	10/10
1034 0000	A6021	4in x 4in	10/10

**NOTE:**

- HCPCS codes contained in this document are provided for informational purposes only and represent no statement, promise, or guarantee by Covalon Technologies Ltd. concerning levels of reimbursement, payment, or charge.
- As with any medical device, always refer to the full instructions for use prior to application.

Manufactured by:



[www.covalon.com](http://www.covalon.com)

Distributed by:



For ordering information call 800-243-2294 or visit [www.hartmannusa.com](http://www.hartmannusa.com)